



Cascade Medical Imaging, LLC

The information below will enable you to view images and reports from the CMI web site in a secure environment.

FOLLOW THESE STEPS to ENTER THE WORLD of FILM-FREE EFFICIENCY:

WEB AMBASSADOR INSTALL – before downloading the software be sure to close any open programs you are using. You will also need to check the encryption level of your Internet browser – it needs to be 128 bit. With Internet Explorer you can check this by going to the tool bar, clicking on “Help” then “About Internet Explorer” –update encryption as needed. Also check to be sure you have Acrobat Reader loaded on your PC. If you don’t have it, you can download it from the Internet. Minimum PC requirements are audio with ***cable Internet***, 128 MB RAM, Windows operating system 2000 or XP and an 800 x 600 Truecolor monitor. You will also need to read, sign and FAX a Confidentiality Agreement to us so we can set-up an account for you and issue a user name and password. Confidentiality agreements can be downloaded from our website: www.cmipacs.com.

You are now ready to download the software. Open Internet Explorer and at the browser window type: <https://images.cmipacs.com> and hit enter. You will be prompted for your user name and password – then hit enter. At the bottom of the Cascade Medical Imaging window click [here](#) to download Web Ambassador. At the top of the DR Systems Web Downloads window click here to download [Web Ambassador](#). At the File Download window click [open](#) then follow the Install Shield Wizard prompts. Close Internet Explorer and click on your new desktop icon: DR Web Ambassador.

Normally the CMI data source will automatically be added during the software install. When you log on, next to the “Database” pull-down menu you will see “CMI_DATABASE” as your first choice. If you do not see this, do the following: At the top of the screen, go to “File” and “Add Web Site Data Source.” Enter: <https://images.cmipacs.com> as the URL and click “Go.” You will be prompted to enter your user name and password again. Read the warnings and click the link next to “If you are using DR Systems Web Ambassador.” Answer **yes** to “Would you like to add the data source?” as well as “Would you like to save it to the default Ambassador configuration?” From the database search window, choose CMI_DATABASE – you will be prompted to re-enter your log-in and password. Click on the “Exam Server” tab to view images. ***Call us at 389-XRAY if you need any help.***